

INTENDED USE

The MOP One Step Morphine Test Device (Urine) is a lateral flow chromatographic immunoassay for the detection of Morphine in human urine at the cut-off concentration of 300 ng/mL. This test will detect other compounds, please refer to Analytical Specificity table in this package insert.

SUMMARY

Opioid analgesics comprise a large group of substances which control pain by depressing the central nervous system. Large doses of Morphine can produce higher tolerance levels and physiological dependency in users, and may lead to substance abuse. Morphine is excreted unmetabolized, and is also the major metabolic product of codeine and heroin. Morphine is detectable in the urine for several days after an opiate dose.¹

The MOP One Step Morphine Test Device (Urine) is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of Morphine in urine. The MOP One Step Morphine Test Device (Urine) yields a positive result when Morphine in urine reaches 300 ng/mL. This is the suggested screening cut-off for positive specimens set by the Substance Abuse and Mental Health Services Administration (SAMHSA, USA).

PRINCIPLE

The MOP One Step Morphine Test Device (Urine) is an immunoassay based on the principle of competitive binding. Drugs which may be present in the urine specimen compete against the drug conjugate for binding sites on the antibody.

During testing, a urine specimen migrates upward by capillary action. Morphine, if present in the urine specimen below 300 ng/mL, will not saturate the binding sites of the antibody coated particles in the test device. The antibody coated particles will then be captured by immobilized Morphine conjugate and a visible colored line will show up in the test line region. The colored line will not form in the test line region if the Morphine level is at or above 300 ng/mL, because it will saturate all the binding sites of anti-Morphine antibodies.

A drug-positive urine specimen will not generate a colored line in the test line region because of drug competition, while a drug-negative urine specimen or a specimen containing a drug concentration less than the cut-off will generate a line in the test line region. To serve as a procedural control, a colored line will always appear at the control line region indicating that proper volume of specimen has been added and membrane wicking has occurred.

KIT COMPONENTS

Individually packed Test Devices

Each device contains a strip with colored conjugates and reactive reagents pre-spread at the corresponding regions.

Disposable pipettes

For adding specimens use.

Package insert

For operation instruction.

MATERIALS REQUIRED BUT NOT PROVIDED

Specimen collection container

For specimens collection use.

Timer

For timing use.

PRECAUTIONS

- For in vitro diagnostic use only. Do not use after the expiration date.
- The test device should remain in the sealed pouch until use.
- All specimens should be considered potentially hazardous and handled in the same manner as an infectious agent.
- The used test device should be discarded according to federal, state and local regulations.

STORAGE AND STABILITY

- The kit should be stored at 2-30°C until the expiry date printed on the sealed pouch.
- The test must remain in the sealed pouch until use.
- Do not freeze.**
- Cares should be taken to protect components in this kit from contamination. Do not use if there is evidence of microbial contamination or precipitation. Biological contamination of dispensing equipments, containers or reagents can lead to false results.

SPECIMEN COLLECTION AND STORAGE

- The urine specimen must be collected in a clean and dry container. Urine collected at any time of the day may be used. Urine specimens exhibiting visible particles should be centrifuged, filtered, or allowed to settle to obtain clear specimen for testing.
- Collected urine specimens must be put in clear and dry containers.
- Perform the testing immediately after the specimen collection. Do not leave the specimens at room temperature for prolonged periods. Specimens may be stored at 2-8°C for up to 48 hours. For long term storage, specimens should be kept below -20°C.

- Bring specimens to room temperature prior to testing. Frozen specimens must be completely thawed and mixed well prior to testing. Avoid repeated freezing and thawing of specimens.
- Pack the specimens in compliance with applicable regulations for transportation of etiological agents, in case they need to be shipped.

PROCEDURE

Bring tests, specimens and/or controls to room temperature (15-30°C) before use.

- Bring the pouch to room temperature before opening it. Remove the test device from the sealed pouch and use it as soon as possible.
- Place the test device on a clean and level surface. Hold the dropper vertically and transfer 3 full drops of urine (approx. 100 µL) to the specimen well (S) of the test device, and then start the timer. Avoid trapping air bubbles in the specimen well (S).
- Wait for the colored line(s) to appear. **Read results at 5 minutes.** Do not interpret the result after 10 minutes.

INTERPRETATION OF RESULTS

POSITIVE RESULT:



Only one colored band appears in the control region (C). No apparent colored band appears in the test region (T).

NEGATIVE RESULT:



Two colored bands appear on the membrane. One band appears in the control region (C) and another band appears in the test region (T).

INVALID RESULT:



Control band fails to appear. Results from any test which has not produced a control band at the specified reading time must be discarded. Please review the procedure and repeat with a new test. If the problem persists, discontinue using the kit immediately and contact your local distributor.

NOTE:

- The intensity of the color in test region (T) may vary depending on the concentration of aimed substances present in the specimen. Therefore, any shade of color in the test region should be considered negative. Besides, the concentration level can not be determined by this qualitative test.
- Insufficient specimen volume, incorrect operation procedure, or performing expired tests are the most likely reasons for control band failure.

QUALITY CONTROL

A procedural control is included in the test. A red line appearing in the control region (C) is considered an internal procedural control. It confirms sufficient specimen volume, adequate membrane wicking and correct procedural technique. Control standards are not supplied with this kit; however, it is recommended that positive and negative controls be tested as good laboratory testing practice to confirm the test procedure and to verify proper test performance.

LIMITATIONS

- The MOP One Step Morphine Test Device (Urine) provides only a qualitative, preliminary, analytical result. A secondary analytical method must be used to obtain a confirmed result. Gas chromatography/mass spectrometry (GC/MS) is the preferred confirmatory method.^{2,3}
- It is possible that technical or procedural errors, as well as other interfering substances in the urine specimen may cause erroneous results.
- Adulterants, such as bleach and/or/album, in urine specimens may produce erroneous results regardless of the analytical method used. If adulteration is suspected, the test should be repeated with another urine specimen.
- A positive result indicates presence of the drug or its metabolites but does not indicate level of intoxication, administration route or concentration in urine.
- A negative result may not necessarily indicate drug-free urine. Negative results can be obtained when drug is present but below the cut-off level of the test.
- Test does not distinguish between drugs of abuse and certain medications.

PERFORMANCE CHARACTERISTICS

Accuracy

A side-by-side comparison was conducted using the MOP One Step Morphine Test Device (Urine) and a leading commercially available MOP rapid test. Testing was performed on 341 clinical specimens. Ten percent of the specimens employed were either at -25% or +25% level of the cut-off concentration of 300 ng/mL Morphine. Presumptive positive results were confirmed by GC/MS. The following results were tabulated:

MOP One Step Test Device	Method	Other MOP Rapid Test		Total Results
	Results	Positive	Negative	
	Positive	159	0	
	Negative	0	182	182
Total Results		159	182	341
% Agreement with this Rapid Test Kit		100%	100%	100%

When compared to GC/MS at the cut-off of 300 ng/mL, the following results were tabulated:

MOP One Step Test Device	Method	GC/MS		Total Results
	Results	Positive	Negative	
	Positive	159	10	
	Negative	0	172	172
Total Results		159	182	341
% Agreement with GC/MS Analysis		100%	94.5%	97.1%

Analytical Sensitivity

A drug-free urine pool was spiked with Morphine at the following concentrations: 0 ng/mL, 150 ng/mL, 225 ng/mL, 300 ng/mL, 375 ng/mL and 450 ng/mL. The result demonstrates >99% accuracy at 50% above and 50% below the cut-off concentration. The data are summarized below:

MOP Conc. (ng/mL)	Percent of Cut-off	n	Visual Result	
			Negative	Positive
0	0	20	20	0
150	-50%	20	20	0
225	-25%	20	20	0
300	Cut-off	20	12	8
375	+25%	20	2	18
450	+50%	20	0	20

Analytical Specificity

The following table lists compounds that are positively detected in urine by the MOP One Step Morphine Test Device (Urine) at 5 minutes.

Compound	Concentration (ng/mL)
Codeine	300
Ethylmorphine	6,250
Hydrocodone	50,000
Hydromorphone	3,125
Levoalphamol	1,500
6-Monoacetylmorphine	400
Morphine	300
Morphine 3-β-D-glucuronide	1,000
Norcocodeine	6,250
Normorphine	100,000
Oxycodone	30,000
Oxymorphone	100,000
Procaine	15,000
Thebaine	6,250

Index or Symbols

Attention, see instructions for use	Tests per Kit	Manufacturer
For in vitro	Use by	Do not reuse
Store between 2-30°C	LOT Lot Number	REF Catalogue Number

